









## Kurz-Gebrauchsanweisung / Short instruction



Mischungsverhältnis / Mixing ratio						
• Pulver [g] powder [g]	100	200	400	500	600	
• Liquid [ml]	20	40	80	100	120	

Konzentration / Concentration				
500 g	60%	70%	80%	90%
Liquid: BegoSol® K	60	70	80	90
Aqua dest.	40	30	20	10

Modell / model (Silikondublierung / Siliconeduplication)	
70%	
Muffel / ring	
70%	

Rührzeiten / Mixing-duration	
Vorspateln / premixing	
15 s	
Anmischen / mixing	
60 s	

Abbindezeit / Setting time	
Modell / model (Silikondublierung / Siliconeduplication)	
30–45 min	
Muffel / Ring konv. / conv.	
20 min	

Einlegezeit / Insertion time	
Entformen / remove	
20 min	
Shock heat [900–950°C]	
20 min	
Max.: Temperatur / Temperature: 900–950 °C	

Artikelnummern / Reference number		
REF 54910	18 kg	72 x 250 g
REF 54911	18 kg	60 x 300 g
REF 54912	6 kg	20 x 300 g
REF 51120	1 Liter	Flasche / bottle (BegoSol® K)
REF 51121	5 Liter	Kanister / canister (BegoSol® K)